# Coltolux® LED

## Curing Light

### TABLE OF CONTENTS

<b>GB</b>	Owner's Guide . . . . .	3
<b>D</b>	Benutzerhandbuch . . . . .	7
<b>F</b>	Guide d'utilisation . . . . .	12
<b>E</b>	Manual de usuario . . . . .	17
<b>I</b>	Guida per l'utente . . . . .	22
<b>NL</b>	Gebruiksaanwijzing . . . . .	27
<b>S</b>	Bruksanvisning . . . . .	32
<b>DK</b>	Betjeningsvejledning . . . . .	36
<b>FIN</b>	Käyttöohjeet . . . . .	40
<b>P</b>	Guia do Utilizador . . . . .	44
<b>G</b>	Οδηγίες χρήσης . . . . .	48
<b>NK</b>	Bruksanvisning . . . . .	53

**coltène**   
**whaledent**®

## Congratulations...

...on your purchase of the Coltolux® LED Curing Light. Coltolux® LED is designed for intra- and extra-oral polymerization of visible-light-cured dental materials with Camphorquinone (CQ) Photoinitiators. See Figure 1 and Figure 2 for component recognition.

### Fig. 1.

- a. Handpiece
- b. Handpiece Charger Base
- c. Activation Button

### Fig. 2.

- d. Voltage/Serial Number Label
- e. Power Cord with Power Supply
- f. Plug Adapters (Export Only)
- g. Screwdriver (Export Only)
- h. Test Block
- i. Light Shields
- j. Barrier Sleeves

Please take special note of the safety precautions below.

### I. SAFETY PRECAUTIONS

A. Observe the following precautionary procedures before and whenever using the Coltolux® LED Curing Light:

**CAUTION:** Before connecting, read all instructions.

**CAUTION:** Light output from this unit, potentially over 1000 mW/cm<sup>2</sup>, may be a significant increase compared to other polymerization devices; therefore, when using this curing light:

- Determine curing time of your material using the enclosed test block.

- Adjust curing techniques in accordance with the increased energy.

**WARNING:** Do not substitute the light lens from other curing lights. Use only the light lens supplied for this unit. Never use components not designed for use with this device since this may impair performance. Do not refrigerate or put in freezer to cool.

**CAUTION:** Persons having a history of photosensitive reactions or who are using photosensitizing drugs should not be exposed to light from this unit.

B. All high intensity lights may cause retinal damage with prolonged exposure to the eyes, therefore we urge you observe these cautionary statements:

**CAUTION:** Never look directly into the light emitted from the light lens or reflected from tooth or other surfaces.

- Always use light shields (Cat. No. C7944) available for light curing or use eye protection goggles (Cat. No. C7988) that block light in the blue spectral range. Sunglasses are not adequate protection.
- Before each use, inspect the light lens (*Section VIII*).
- Patients, operators and dental technicians should always use proper eye protection.

**CAUTION:** The red indicator light on the charger indicates a warning that the battery may not have sufficient charge for proper curing operation and that action must be taken to charge the battery.

## II. UNPACKING

- Before unpacking, examine the carton for external damage, and if severe, contact your shipper/carrier.
- Open carton and carefully remove the unit and accessories. Make sure supply voltage is within the range specified on the power supply (*Fig. 3*). If not, do not operate this unit; contact your dealer or Coltène/Whaledent Inc.

## III. SET-UP AND ASSEMBLY

- Coltolux® LED is designed as a cordless handpiece unit with a separate charger base. Place the charger base on a smooth flat surface in an area that is dry and free of dust, is convenient and has an outlet within easy reach of the supplied power cord. Export units include additional plug adapters. Install the appropriate plug adapter onto the power supply (*Fig. 2*). To change the plug adapter, remove screw, slide existing plug adapter out of power pack and replace with desired plug adapter, insert screw and tighten securely (*Fig. 6*). Plug the power cord into the wall outlet and then into the charger base. A green light on the charger base indicates power is available to charge the handpiece.
- Insert the handpiece into the charger base. When the handpiece is being charged, the light on the charger base will be red. When the handpiece is fully charged and ready, the red light on the charger base will turn green.

The initial charging of the handpiece will require approximately 3 hours to fully charge.

- When the handpiece is removed from the charger base the small light above the On/Off button on

the handpiece will be green, indicating the unit is ready for use. If the light does not light up green, see **Section VI Troubleshooting**.

**CAUTION:** Never modify the power cord.

**WARNING:** Explosion hazard. Do not use in the presence of flammable anaesthetics.

#### IV. FUNCTION CHECK

##### A. Activate/Deactivate Light

Depressing the On/Off button on the handpiece activates or deactivates the curing light (**Fig. 4**). Do not hold the button continuously throughout the curing cycle. During operation an audible beep will be emitted every 10 seconds. The unit shuts off automatically after 60 seconds.

##### B. Measuring the Light's Effectiveness

To ensure complete polymerization at all times, it is critical to periodically test the light intensity output.

**IMPORTANT:** Test the Coltolux® LED Curing Light before first use, and at least once every other week thereafter (depending on usage) using a light meter and/or the provided test block.

1. Enter your readings on the "Meter Reading Chart" at the end of this Owner's Guide.

To test the Coltolux® LED, use Coltolux® Light Meter (sold separately – Cat. No. C7900WHT).

Push the light shield away from the tip of the light lens so it does not interfere with the sensor. To take the reading, hold the handpiece in front of the meter (**Fig. 5**). Place

the light lens on top of the sensor located on the front of the meter. Activate the light, and allow ten (10) seconds for stabilization of the meter.

**IMPORTANT:** Place the light lens on the center of the sensor, and hold steady while taking the reading.

To take another reading, repeat the steps outlined above.

##### 2. Meter Display

A reading of 300 mW/cm<sup>2</sup> or higher indicates that the light output is acceptable for the polymerization of material up to a 2.0 mm depth. Follow the dental material manufacturer's instructions for recommended exposure time, and check using the supplied test block.

A reading below 300 mW/cm<sup>2</sup>, or a reduction of intensity of the light output in comparison to the original value, indicates that corrective action should be taken. (See **Section VI, D Troubleshooting**.)

#### V. CLINICAL APPLICATION

**CAUTION:** Always test restorative materials with this LED light by using the supplied test block.

##### A. Polymerization

- Determine the exposure time in accordance with the dental material manufacturer's instructions or by using the test block provided.
- Remove the charged handpiece from the charger base; position the light shield; slide the barrier sleeve over the light and light shield; direct the tip of the light lens as close to the restorative material as possible without touching.

- Activate Coltolux® LED with a momentary push of the button. Avoid contact with unpolymerized restorative material, glass ionomer, liquid resin or bonding agent because direct contact will cause these materials to adhere to the light lens. This will block the light transmission and result in improper polymerization.

##### B. Over-Heating Protection

Coltolux® LED is safeguarded against overheating.

1. A thermal overheat protector will be activated when the unit is in continuous operation for longer than three (3) minutes. The Blue light output will shut off. The thermal overheat protector will automatically reset once the unit is cooled and ready for use. Reactivate the Coltolux® LED light by a momentary push of the button.

#### VI. TROUBLESHOOTING

- A. If the light on the charger base does not illuminate:
  - Check to see that the power cord is securely plugged into the charger base and the wall outlet.
  - Ensure the wall outlet has power by plugging a lamp into it.
- B. If the On/Off button on the handpiece does not activate the blue LED curing light and the green light on the handpiece is not lit (and handpiece is not in base):
  - The unit may be overheated. Allow unit to cool and try again.
  - The unit may be discharged and needs charging. Replace handpiece in the charger base. If the charger base LED is red, handpiece is being charged. If it is not, see "A" above.

**C.** If the On/Off button on the handpiece does not activate the curing light and the green light above the button is illuminated, unit requires service. Contact your dealer or Coltène/Whaledent Inc.

**D.** If your light registers a reading below 300 mW/cm<sup>2</sup> on a light-meter, take the following steps:

- Check the light lens.

The light lens should be checked daily. Replace lens with Coltène/Whaledent Inc. Light Lens Cat. No. C7974 if material has cured to the lens.

- If the intensity of the light output is still under 300 mW/cm<sup>2</sup>, extend the curing time according to the material's manufacturer recommendation until the Coltolux® LED light can be serviced. Contact your dealer or Coltène/Whaledent Inc. for service.

## VII. REPLACING THE LIGHT LENS

**IMPORTANT:** The Coltolux® LED Light Lens is easily scratched and must be handled with care at all times.

**CAUTION:** Only use Coltène/Whaledent's Light Lens Cat. No. C7974. Do not use other manufacturer's light lens with this product.

**CAUTION:** Remove the handpiece from the charger base before attempting to replace the light lens. Do not activate the handpiece.

- Remove the light lens by rotating the lens counterclockwise until the lens disengages from the handpiece.

**CAUTION:** Do not touch the LED under the light lens as hand oil contaminates the LED and can compromise performance.

## VIII. INSPECTING THE LIGHT LENS

- Inspect the light lens before each use to insure that restorative material has not accidentally cured to the lens. Replace lens with Coltène/Whaledent Inc. Light Lens Cat. No. C7974 if material has cured to the lens.

**CAUTION:** Never look directly into the light emitted from the light lens or reflected from tooth or other surfaces.

## IX. INSPECTING THE CHARGER BASE POWER CORD

- Periodically inspect the charger base power cord for frayed areas or broken insulation.

## X. CLEANING

- All external components may be cleaned by wiping with a soft cloth moistened with isopropyl alcohol.

**IMPORTANT:** Do not use petroleum-based solvents, iodophors, or phenolic-based products on the handpiece or charger base. (Iodophors and phenolics can stain the surface of the unit and damage the plastic housing.)

## XI. INFECTION CONTROL

- To comply with infection control procedures the light shield is to be placed on the handpiece first, then slide the barrier sleeve over the handpiece and shield. A new barrier sleeve should be used for each patient.

## XII. SPECIFICATIONS

### A. Electrical Safety

Coltolux® LED is a tested and listed device under UL and CSA standards.

This product fulfills the requirements concerning electromagnetic compatibility in accordance with EMC directive 93/42/EEC.

## B. Dimensions (approximate)

### Handpiece

Length: 22.1 cm / 8.7"

Probe Dia: 1.22 cm / .48"

Grip Dia: 2.5 cm / 1.00"

### Charger Base

Height: 7.4 cm / 2.9"

Width: 6.4 cm / 2.5"

Depth: 8.9 cm / 3.5"

## C. Operating Voltage/Power

### Power Supply

Input: 100 – 240 VAC, 50 – 60 Hz

Output: 5 VDC, 6 watts

Class: II

### Handpiece

Operating Voltage: 4.2 – 3.0 VDC

## D. Environmental Conditions

Coltolux® LED is designed to operate in ambient temperatures from 50°F – 104°F (10°C – 40°C), relative humidity 80% at 77°F (25°C).

**CAUTION:** To maximize patient comfort, do not operate handpiece for more than one minute continuously if the ambient temperature is above 30° C.

## E. Classifications & Symbols

Coltolux® LED is classified as IEC60601-1 Class II, Type BF, for protection against electric shock.

IEC Symbol  
Publication: 878-02-03  
Type BF Equipment

**IEC Symbol**  
**Publication: 348**  
**Attention: Consult**  
**accompanying documents**

**IEC Symbol**  
**Publication: 416**  
**Class II Equipment**

### Accessories

#### **ONLY USE AUTHORIZED Coltène/Whaledent Accessories**

- C7900WHT** Light Meter
- C7944** Disposable Light Shields  
 for Light (100 ea.)
- C7957** Barrier Sleeves (1000 ea.)
- C7974** Light Lens (25 ea.)
- C7988** Goggles

### **XIII. INSTRUCTIONS FOR USE**

Coltolux® LED Curing Lights have been developed specifically for use in dentistry and are intended to be operated only by dental professionals in accordance with the instructions contained in this guide. However, notwithstanding anything contained herein, the user shall at all times be solely responsible for determining the suitability of the product for the intended purpose and the method of its use. Any guidance on application technology offered by or on behalf of the manufacturer, whether written, verbal or by demonstration, shall not relieve the dental professional of his/her obligation to control the product

and to make all professional judgements regarding its use.

Coltolux® LED Curing Lights are manufactured to impeccable standards. These products are guaranteed in accordance with the terms of the written Certificate of Limited Warranty which accompanies each product.

**Except for the warranties specifically set forth in the Certificate of Limited Warranty, Coltène/Whaledent Inc. provides no warranties or guarantees covering this product, expressed or implied, including, without limitation, any warranties as to merchantability or fitness for purpose. Please refer to the Certificate of Limited Warranty for warranty terms and conditions.**

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**Coltène / Whaledent Ltd.**